



...Beyond Illumination

It illuminates.  
It irrigates.  
It aspirates.  
It retracts.

## Multi-Functional Surgical Illuminator

### VersaLight™

from Lumitex MD is a single-use multi-functional surgical device that illuminates, irrigates, aspirates, and provides moderate blunt retraction. VersaLight allows the surgeon easily and efficiently to control four vital operating field functions, while offering superior visualization and minimizing the number of implements used in the surgical site.

Using Lumitex's patented MicroLens® lighting technology, VersaLight provides a brighter, cooler light that doesn't heat up – or shut down – during surgery. Its bright shadowless light floods the field, helping to identify landmarks, locate bleeders, transilluminate tissue, and avoid complications.

VersaLight can be inserted directly into deep cavities, small openings, under flaps, and in lateral margins. It helps to access, visualize, irrigate, and clear the surgical site, while saving operative time, space, and cost. Designed to attach easily to existing OR equipment, it requires no capital investment, and provides a lower cost-per-use.




Optional extended tip provides (need info)



# VersaLight™: Four Key Ways It Takes You Beyond Illumination.

## 1. IT ILLUMINATES.

VersaLight's patented Lumitex lighting technology floods the operative field with bright, shadowless light, as if a fluorescent light were inside the surgical field. Because it can also function as a blunt retractor, it provides a clearer view, further improving visualization. The specially designed light cable attaches to a standard OR light source with an ACMI connection.

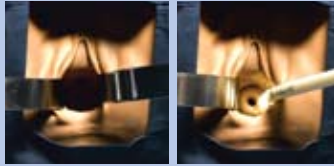

Without VersaLight

With VersaLight



VersaLight connects easily to standard OR equipment.

## 2. IT IRRIGATES.

VersaLight irrigates at the push of a button, providing a controlled stream of fluid on demand. A simple luer-lock connection on the proximal end attaches to standard hospital tubing, providing sterile irrigation from a sterile solution bag with a pressurized cuff or any standard infusion pump.



IFU: Monitoring and maintaining pressure for optimal results (Maximum 300 mmm)

## 3. IT ASPIRATES.



By placing a fingertip over the conveniently located aspiration hole, surgeons can easily control the strength of suction. A quick-connect on the proximal end attaches it to standard OR suction tubing.

## 4. IT PROVIDES MODERATE BLUNT RETRACTION.

VersaLight's horizontally seamed design enables it to withstand approximately ten pounds of retraction force without breaking, making it useful to reposition tissue for better access and visualization.



Distributed by:

© 2006 Lumitex, Inc. Lumitex is a registered trademark of Lumitex, Inc. VersaLight is a registered trademark of Lumitex, Inc. Lumitex products are protected by U.S. and foreign patents and patents pending. ISO 13485 Reg. No. SX 60013530 0001 CE Reg. No. HD 60008817 0001 Bulletin 006266 Rev. E 10/08

CE ISO

## PRODUCT FEATURES

### Four functions, one instrument.

Saves operative time, space, and cost. Minimizes number of instruments in sterile field.

### Bright, cool, shadowless light.

Better illumination helps identify surgical landmarks, locate bleeders, transilluminate tissue, and avoid complications. Does not heat, dry, or damage tissue on contact.

### Ergonomic design.

Easy to use. Lightweight; counter-balanced to work well with any hand grip: overhand, pencil or fingertip. Thinner tip. Easy connections. Reduces hand fatigue.

### Easy irrigation; channeled aspiration tip.

Helps avoid infection. Minimizes "tissue grab".

### Moderate blunt retraction.

Helps reposition tissue for better access and visualization. Perfect for soft tissue retraction in inaccessible places.

### Sturdy longitudinal design.

Increased durability and strength. Resists cracking and breaking.

## LOW COST

### Less expensive disposable than the competition.

### Requires no capital cost.

Uses common OR light sources. Fits existing OR tubing and equipment.

## ORDERING INFORMATION

For more information on VersaLight call (800) 969-5483 or talk to your Lumitex MD representative. You may also visit us at [www.LumitexMD.com](http://www.LumitexMD.com) and click on the VersaLight tab at the top.

VersaLight Part #: VSG211

VersaLight Extended Tip Part #: VSX222

VersaLight ACMI Cable Part #: 006241

VersaLight Storz Cable Part #: 006422

VersaLight Wolf Cable Part #: 006423

VersaLight Olympus Cable Part #: 006424

 **Lumitex MD™**  
LUMITEX MEDICAL DEVICES, INC.

Manufactured by Lumitex, Inc.  
8443 Dow Circle  
Strongsville, OH 44136  
Phone: (440) 243-8401 • (800) 969-5483  
Fax: (440) 243-8402  
E-mail: [medical@lumitex.com](mailto:medical@lumitex.com)  
[www.versalight.info](http://www.versalight.info)